

Welcome to your sampling kit

With this kit you can collect and return the samples needed for your recommended tests.

The results of this STI test will show whether a bacteria or virus is present. It's not a clinical diagnosis of a condition and should not replace advice from your doctor. If you have any symptoms of an STI or concerns regarding your sexual health, you should speak with your GP or visit a local sexual health clinic.


Name | Reference

When did you collect your sample?

D	D	/	M	M	/	Y	Y	Y	Y
---	---	---	---	---	---	---	---	---	---

Your samples will be tested for

Please detach this form and return it in the kit box with your samples

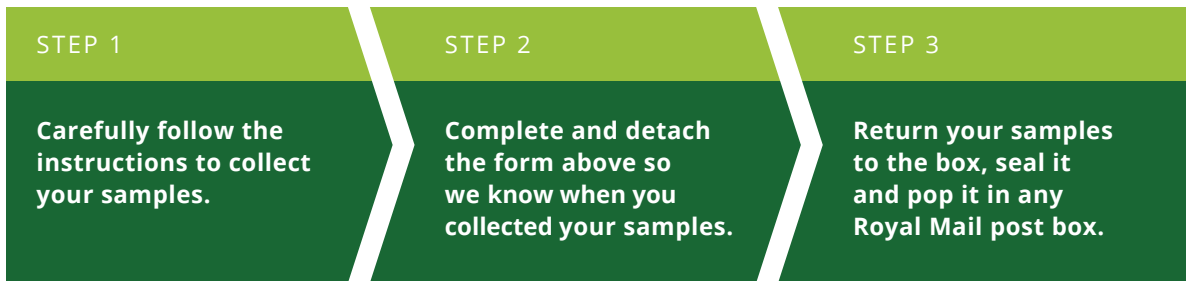

You can expect your results within 2-5 working days of your sample arriving back at the lab.

Find them at www.onlinedoctor.lloydspharmacy.com

Returning your samples

Once you're ready to return your samples, place them along with your completed form back into the kit box.

Close it securely by inserting the two large tabs on the box lid into the slots, pressing down firmly to lock them into place. Apply the clear security seal provided or add tape for extra security.

Return postage is not required, just pop the kit box into any Royal Mail post box. Find your nearest post box by scanning this QR code.

Collecting your vaginal swab

What you will need



Vaginal swab

- 1 Wash your hands and unpack the kit on a clean surface, ideally in a bathroom.
- 2 Remove the swab from its tube. **Avoid laying it down or touching its tip.**
- 3 Carefully insert the swab half a finger length into your vagina.



- 4 Now rotate the swab around the inside of your vagina 5–10 times.
- 5 Return the swab to the tube and close tightly.

